



BODE SCIENCE CENTER

Research for infection protection.
www.bode-science-center.com



Wipes dispensers without foil bag: automated reprocessing procedure

The reprocessing procedure may be used for all wipes dispenser systems without foil bag – independent of whether they are filled with surface disinfectants based on surface-active substances without aldehyde or surface disinfectants based on aldehydes or alcohols.

Before reprocessing, it is imperative to remove and discard any fleece roll and liquid residue.

Automated reprocessing procedure



In professional machines only.



At 60 °C and with an exposure time of 5 minutes.



Reprocessing with mildly alkaline / highly alkaline or without cleaner.



Refill with new fleece roll and use-solution.