

Handling alcohol-based surface disinfectants



Pre-soaked Tissues

For immediate use.



Ready-to-use Solution*

Apply liquid with a disposable wipe.



Ready-to-use Foam

Spray foam onto the surface or onto a disposable wipe.

Application

CORRECT:

- Remove visible dirt from surfaces before disinfection
- Wet surfaces evenly with liquid
- Wipe with evenly exerted pressure
- Wait until surfaces have dried completely
- Switch off ignition sources and ventilate well, as products with high alcohol content are highly flammable
- After use, dispose of wipes in household waste
- Close packaging immediately after use

WRONG:

- Dilute disinfectant
- Disinfect floors or larger surfaces with alcohol-based products
- Use for hand disinfection
- Damp wiping is enough
- Wipe the surfaces dry
- Allow disinfectant to get inside the electronic device

Products

for rapid disinfection against bacteria, yeasts and enveloped viruses, e.g. coronaviruses



Products with high alcohol content

Only use on alcohol-resistant surfaces (e.g. not on acrylic or plexiglass)

Pre-soaked Tissues:

- Bacillo® Tissues
- Bacillo® AF Tissues

Ready-to-use Solution:

- Bacillo® AF



Products with low alcohol content

Can also be used on sensitive plastics, acrylic and plexiglass as well as monitors, displays, keyboards, etc.

Pre-soaked Tissues:

- Bacillo® 30 Sensitive Tissues

Available as of January 2022*

Ready-to-use Foam:

- Bacillo® 30 Sensitive Foam



* The release date may vary in different countries and regions. Contact your local distribution partner for more information.
GER, AT, CH, CZ, LU, LT, LV, SI, SK, GB, SE, EE, PL, IE: January onwards (Status Nov. 21); ES, PT, HU, GR, UA: March onwards (Status Nov. 21)

* German jurisdiction: max. 50 ml per m² of surface in accordance with TRGS 525



Research for
infection protection
hartmann-science-center.com

Download
Poster:



Use surface disinfectants safely. Always read the label and product information before use.

CE 0482